





Application Techniques: Dressing Secural

Picture Frame Technique

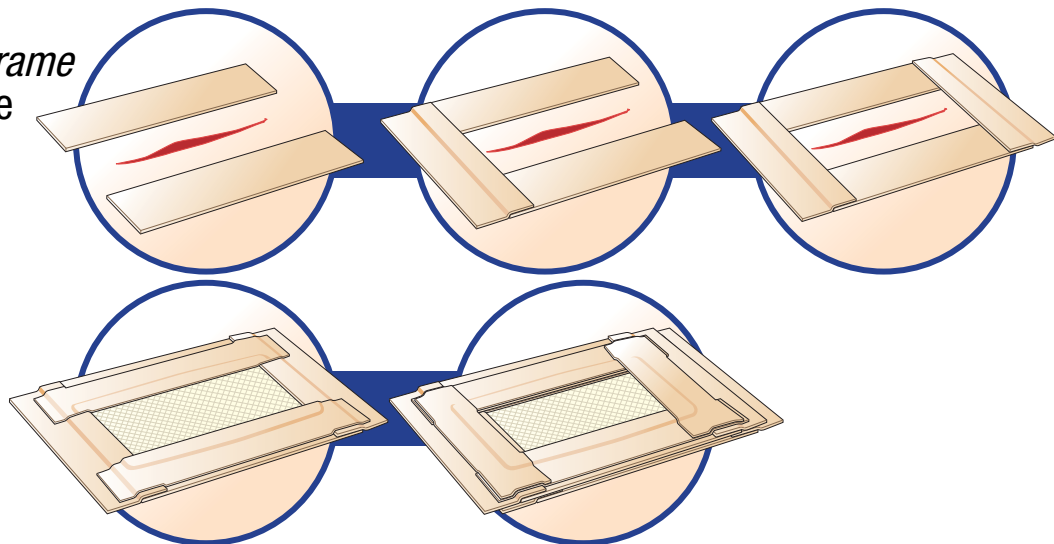

- A platform of tape may be placed around wound before the dressing is applied.
- Place dressing over wound.
- Place tape strips over each dressing edge, applying onto platform, smoothing from centre out.

Strip Technique

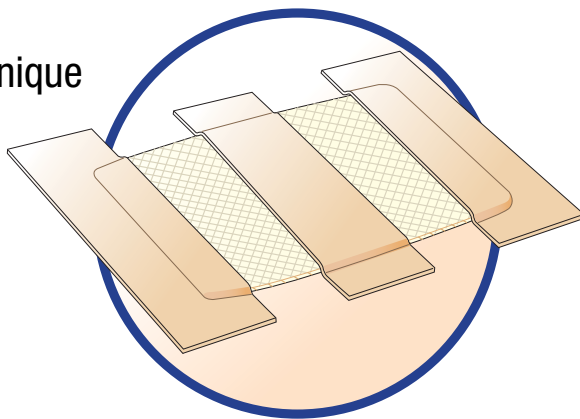

Parallel strips of tape are placed across the dressing leaving open areas between the strips.

Hints and Tips

3M™ Coban™ Self-Adherent Bandage may be used to secure dressings with light tension around limbs or digits.

3M Health Care

3M House
Morley Street
Loughborough
Leicestershire LE11 1EP
Telephone (01509) 611611
Facsimile (01509) 237288
Website: <http://www.3mhealthcare.co.uk>

3M Ireland

3M House, Adelphi Centre
Dun Laoghaire
Co Dublin
Ireland
Telephone (01) 280 3555
Facsimile (01) 280 3509

3M and Tegaderm are trademarks of the 3M Company.
© 3M Health Care Limited 2008.

